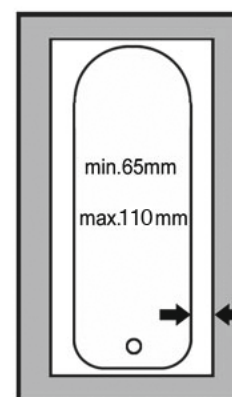
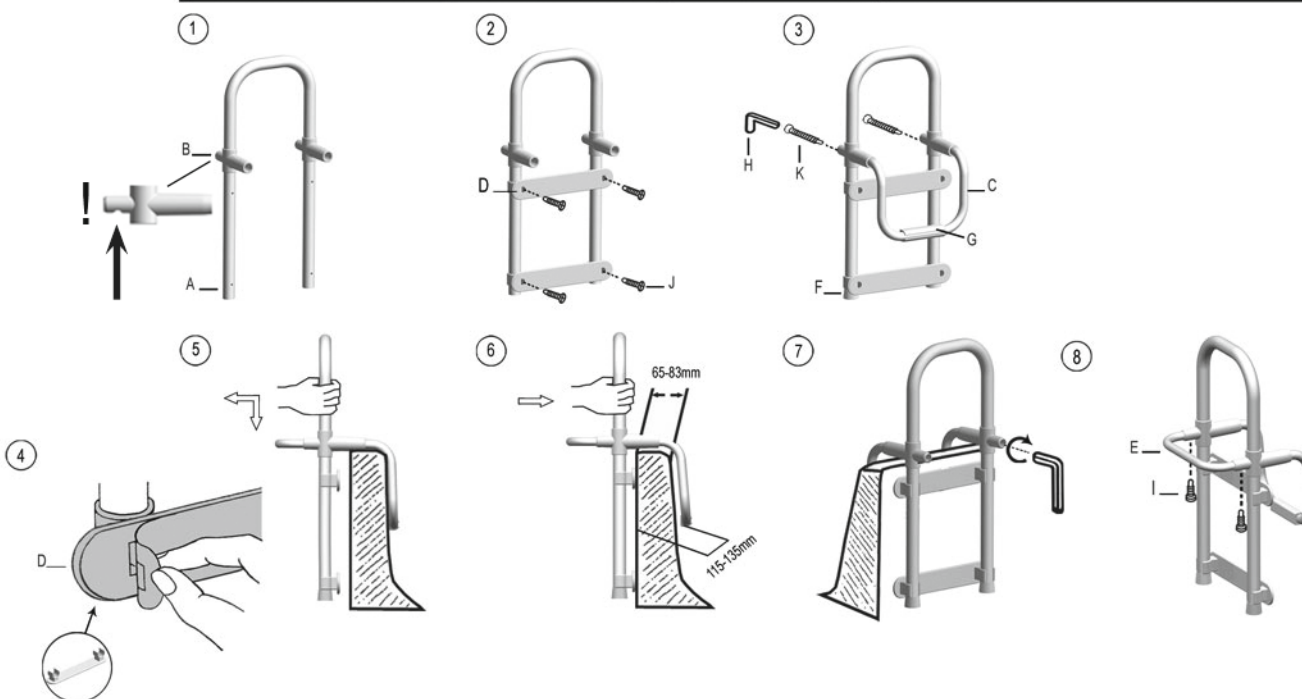
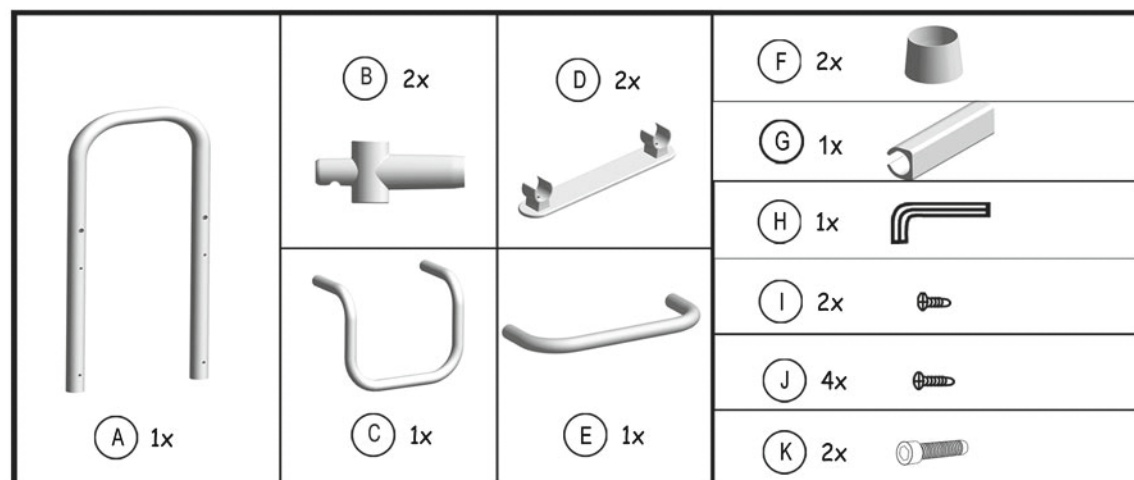
DE Montageanleitung
GB Assembly Instructions
F Instructions de montage
E Instrucciones de montaje
I Istruzioni per il montaggio
NL Montage-instructie

P Instruções de montagem
DK Montagevejledning
S Monteringsanvisning
PL Instrukcja montażu
GR Οδηγία μοντάζ
CZ Návod k montáži

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© Model 8110*, 81101*

Bath-fixed support aid Areas of use:

This product may only be used in private households! Suitable for all conventional bathtubs. The enclosed mounting elements must be used as described in the installation instructions.

CAUTION!

Only suitable for bathtubs without hollow walls! The outer wall of the bathtub must be tiled.

Important note regarding 7:

The plastic stopper G must be firmly seated flush with the inner wall of the bathtub. There must be no gap between the inner wall and the plastic stopper. Once this has been ensured, the two screws K must be additionally turned 6 times!

Safety instructions:

Stick the adhesive strips (located on the retainers D) to the edge of the bathtub. The edge must be thoroughly cleaned prior to adhesion, and must then be free from water residues, grease and chemical substances! The adhesive must be allowed to dry for 24 hours after attaching the bath-fixed support aid! During this period the bath-fixed support aid must not be subjected to any loads! The screws K must be tightened while applying force, until the bracket C with the stopper G is very firmly seated against the inner wall of the bathtub! Please check that the bathtub is dry before using the grab handle. Not suitable for use in rehabilitation facilities and for the medical care of persons. Only use the grab handle in the way described in the instructions for use. Accidents resulting in serious injuries may occur if the handles are used for incorrect purposes e.g. as gymnastic

equipment or steps. Wenko Wenselaar GmbH & Co KG shall not assume any liability whatsoever in the event of incorrect use!

Maintenance and inspection instructions:

The handle should be checked regularly for stability and damage, and reuse is only permitted after the appropriate repairs have been carried out. Check connecting elements regularly and re-tighten screws if necessary.

Care instructions:

Clean with a damp cloth. Do not use any sandy abrasive agents or scrub sponges for cleaning. Please keep these instructions, as they contain important information.

Exclusions:

Wenko does not assume any liability for damage to persons and property caused by improper mounting, incorrect use of the handles, in particular by non-observance of the warning and safety instructions listed above.

For after-sales service please contact the manufacturer directly:

Wenko-Wenselaar GmbH & Co. KG
 Im Hülsenfeld 10 · 40721 Hilden, Germany
 Tel.: +49 2103 573-0 · Fax: +49 2103 573-190
 service@wenko.de · www.wenko.com

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