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
- Ⓓ Badstützgriff
- ⒼⒷ Bathroom safety handle
- Ⓕ Poignée de sécurité pour salle de bain
- Ⓔ Asidera de apoyo para el baño
- Ⓘ Maniglia di sostegno per bagno
- ⓃⓁ Veiligheidsgreep voor de badkamer
- Ⓟ Pega de apoio para casa de banho
- ⒹⓀ Armstøtte til badeværelser
- Ⓢ Stödhandtag för badrum
- ⓅⓁ Uchwył łazienkowy
- ⒼⓇ Χειρολαβή-στήριγμα
- ⒸⓏ Koupelnová opěrka
- ⓈⓁⒹ Oporni drżaj za kopalnico
- ⓈⓀ Kúpeľňové držadlo
- Ⓗ Fürdőszobai kapaszkodó
- ⓇⓊⓈ Опорная ручка для ванной комнаты
- ⓉⓇ Banyo için destek kolu

- Ⓓ Klappbarer Badstützgriff
- ⒼⒷ Foldable bathroom safety handle
- Ⓕ Poignée de sécurité repliable pour salle de bain
- Ⓔ Asidera de apoyo plegable para el baño
- Ⓘ Maniglia di sostegno ribaltabile per bagno
- ⓃⓁ Opklapbare veiligheidsgreep voor de badkamer
- Ⓟ Pega de apoio rebatível para casa de banho
- ⒹⓀ Opklappelig armstøtte til badeværelser
- Ⓢ Fällbart stödhandtag för badrum
- ⓅⓁ Składany uchwył łazienkowy
- ⒼⓇ Διπλωνόμενη χειρολαβή-στήριγμα
- ⒸⓏ Sklopná koupelnová opěrka
- ⓈⓁⒹ Sklopni oporni drżaj za kopalnico
- ⓈⓀ Sklápacie kúpeľňové držadlo
- Ⓗ Felhajtható fürdőszobai kapaszkodó
- ⓇⓊⓈ Откидная опорная ручка для ванной комнаты
- ⓉⓇ Banyo için katlanabilir destek kolu



Modell:
17964*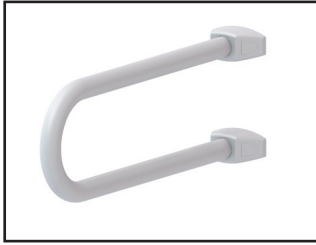

17969*

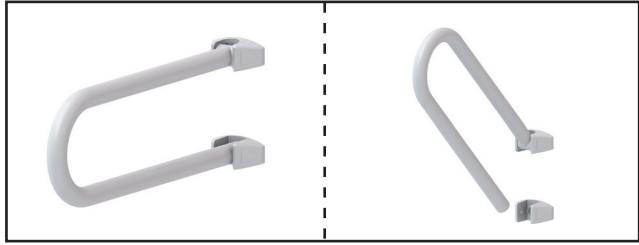17964*

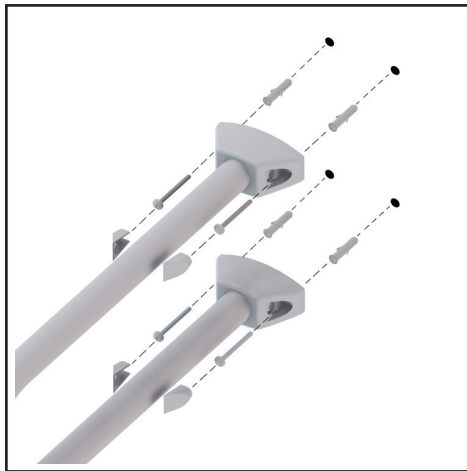17969*

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(D) Montageanleitung	6-7
(GB) Assembly Instructions	8-9
(F) Instructions de montage	10-11
(E) Instrucciones de montaje	12-13
(I) Istruzioni per il montaggio	14-15
(NL) Montage-instructie	16-17
(P) Instruções de montagem	18-19
(DK) Montagevejledning	20-21
(S) Monteringsanvisning	22-23
(PL) Instrukcja montażu	24-25
(GR) Οδηγία μοντάζ	26-27
(CZ) Návod k montáži	28-29
(SLO) Navodilo za montažo	30-31
(SK) Návod na montáž	32-33
(H) Szerelési útmutató	34-35
(RUS) Инструкцид по монтаж	36-37
(TR) Montaj kılavuzu	38-39

- Ⓢ Montageanleitung
- Ⓢ Assembly Instructions
- Ⓢ Instructions de montage
- Ⓢ Instrucciones de montaje
- Ⓢ Istruzioni per il montaggio
- Ⓢ Montage-instructie

- Ⓢ Instruções de montagem
- Ⓢ Montagevejledning
- Ⓢ Monteringsanvisning
- Ⓢ Instrukcja montażu
- Ⓢ Οδηγία μοντάζ
- Ⓢ Návod k montáži

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Tel: +49 2103 573-0 • Fax: +49 2103 573-190

service@wenko.de • www.wenko.de

Operating instructions for Bathroom safety handle/ Foldable bathroom safety handle

Model: 17964* Bathroom safety handle
17969* Foldable bathroom safety handle

Application areas:

Wall-mounted support handles and wall-mounted folding support handles are mainly fitted in sanitary areas on both sides of wash basins and toilets. This product may only be used in private households! Note: Not suitable for use in rehabilitation facilities and for the medical care of persons.

Areas of use:

Only use the support handle in the way described in the operating instructions. Accidents resulting in serious injuries may occur if the handles are used for incorrect purposes e.g. as gymnastic equipment or as steps

Support handles are designed for a maximum body weight of 120kg.

Do not use the handles if you exceed this weight. If the load values are exceeded, permanent damage may occur to the handle or the joint.

Note for model 17969100:

No persons (particularly children) must stand underneath the handle when it is folded down. In case of non-observance, accidents resulting in serious injuries may occur.

A certain vertical and horizontal bending under load is normal. If you should detect any permanent bending of the supporting handle, you should immediately inform the maintenance company or after-sales service.

Installation of the supporting handle:

This installation may only be carried out by qualified technical staff (plumbers).

The perfect and safe function of the supporting handle largely depends on the professional installation on the wall. Incorrectly performed installation can lead to serious injuries. Self-installation of the handle is therefore not recommended. When installation of the supporting handle takes place by qualified technical staff, it must be observed that the attachment surface in the entire area of the wall plate is even (no protruding joints or tiles), the wall structure is suitable for the attachment of the handle and in particular that it has no weak points. Furthermore attention must be paid to the selection of suitable fastening material. Make sure to check and consider the routing of electrical cables and water pipes before drilling. The handle should be checked regularly for stability and damage, and reuse is only permitted after the appropriate repairs have been carried out.

17964* Bathroom safety handle

4 pc. steel screws DIN 7982 (ø5,5 x 70 mm)

4 pc. dowels (ø8 x 50 mm)

2 pc. hexagonal nuts

(self-locking) DIN 985

2 pc. M8 x 50 mm screw DIN 7985

17969* Foldable Bathroom safety handle

4 pc. steel screws DIN 7982 (ø5,5 x 70 mm)

4 pc. dowels (ø8 x 50 mm)

1 pc. hexagonal nuts

(self-locking) DIN 985

1 pc. M8 x 50 mm screw DIN 7985

Assembly instructions:

In the scope of delivery only the fastening material for concrete is included.

Obtain mounting materials for other wall materials via the specialist personnel.

Use the template included in the scope of delivery: ①

② Check wall structure! Drill 8 cm dowel holes, remove mounting template. Knock in dowels and screw the supporting handles in.

Care instructions:

Occasional wiping with a cloth is sufficient.

You can use detergents and soaps as cleaning agents.

Please do not use alkaline, acidic and chlorinated cleaning agents or cleaning agents containing abrasive additives.

Safety instructions:

Check connecting elements regularly and re-tighten screws if necessary. Wenko Wenselaar GmbH & Co KG shall not assume any liability whatsoever for incorrect use! Please keep these instructions, as they contain important information.

Exclusion:

Wenko does not assume any liability for damage to persons and property caused by improper mounting, incorrect use of the handles, in particular by non-observance of the warning and safety instructions listed above.

For after-sales service please directly contact the manufacturer:

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Wenko-Wenselaar GmbH & Co KG
Wenko-Wenselaar GmbH & Co KG · Im Hülsenfeld 10 · D-40721 Hilden