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GB Operating instructions

Regulation (EU) 2016/425, particle-filtering face mask according to EN 149:2001 + A1:2009 for serious risks Cat.III

Note: Disposable breathing equipment is classified according to the categories FFP1, FFP2 and FFP3. Before you read the following information, check which category the breathing equipment belongs to, this is marked on the packaging and the breathing equipment. The user is responsible for making sure that the breathing equipment has the necessary protection level for the type and concentration of soiling in the area where the breathing equipment is to be used.

Intended use

Generally acknowledged accident prevention regulations and enclosed safety instructions must be observed.

Only perform work described in these instructions for use. Any other use is improper. The manufacturer will not assume responsibility for damage resulting from such use.

FFP1:

The FFP1 particle-filtering face mask is for protecting the respiratory system during work with non-toxic dusts and aerosols with a concentration up to 4 times the MAC value.

Area of application:

Sweeping, sanding, cutting and drilling rust, plaster, concrete and masonry

FFP2:

The FFP2 particle-filtering face mask is for protecting the respiratory system against harmful and carcinogenic dust, smoke and aerosols up to 10 times the MAC value.

Area of application:

Insulating materials, hardwood dust, quartz dust, lead dust, welding gas and blue smoke, fungi

FFP3:

The FFP3 particle-filtering face mask is for protecting the respiratory system against solid and liquid particles of toxic and highly toxic substances up to 30 times the MAC value

Area of application:

Carcinogenic and radioactive particles, biologically active aerosols, pharmaceutical industry, heavy metal particles, stainless steel welding, oil mist and other aerosols, asbestos

Instructions for use/risk assessment:

- Put the mask on and check it is airtight before you enter the soiled area.
- Always wear the breathing equipment as long as you are exposed to soiling.
- Use the breathing equipment according to the respective health and safety requirements.
- Dispose of the breathing equipment and use a new one:
 - if the breathing equipment was removed whilst still in a soiled room.
 - if the breathing equipment is soiled to such an extent that you have difficulty breathing and/or the mask is uncomfortable to wear.
 - if the breathing equipment is damaged.
- Leave the soiled area if you feel dizzy or sick, or show any other signs of discomfort.
- The breathing equipment is for one-off use, which means no maintenance is necessary.
- Do not keep and/or use again after use.
- Destroy the breathing equipment after the first time it is used and after 8 hours of use at the latest (maximum for one working cycle). Keep

unused breathing equipment in a closed box and store this in a dry and non-soiled room between -30 °C and +60 °C.

– They do not provide protection against gases and vapours.

How to put the mask on:

Hold the breathing equipment in your hand such that the straps can hang down freely. A Hold the breathing equipment under your chin with the nose part facing outwards. B Take hold of the lower rubber strap and pull it over your head to your neck, below your ears. Holding the breathing equipment on your face with one hand, pull the upper rubber strap over your head to above your ears. C

To make sure the breathing equipment assumes the correct shape (if the breathing equipment is not fitted with the active shape), use your fingertips of both hands to press the metal reinforcement from the bridge to the wing of your nose. D

Checking air-tightness and fit: E

Take both hands, hold these over the breathing equipment and breathe out deeply. This should cause pressure to develop within the breathing equipment. If you feel air escaping round the edges, adjust the metal reinforcement of the breathing equipment again and/or increase the tightness of the rubber straps. Repeat this procedure until you are satisfied with the mask's fit.

Important:

If you notice that the straps are too loose, increase their tension by making small knots in the rubber strap.

Restrictions:

Do not use the breathing equipment or remain in the soiled area,

- if the oxygen content is lower than 17% (UK 19%),
- if the soiling or concentration are unknown or potentially fatal.
- if in particular concentrations exceed the limiting value which is specified by health and safety regulations, or the protective factor × NPF is lower.
- Nominal protective factors: 4.5 for FFP1, 12.5 for FFP

R: Labelling with the letter "R" means that the filters are reusable.

NR: Labelling with the letters "NR" means that the filters are only suitable for single use.

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Instytut Technologii Tekstylnych CERTEX Sp. z o.o. (Ltd.)